



Users Manual

Ver. 1.2.1

New Registration/ Web app · mikoto unit




1. You can create a new account from the New Registration icon on the mikoto home screen, or by selecting [New Registration] on the login screen.

Register from the QR code here.



Sign up QR

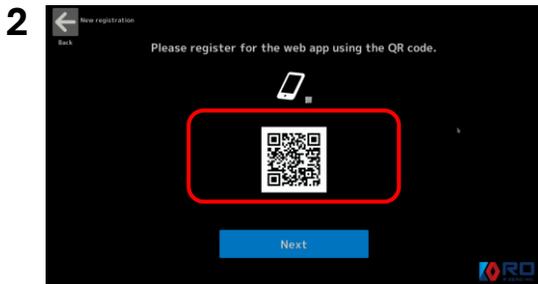

2. Scan the mikoto Web App registration QR code displayed on the mikoto device with your smartphone's camera to sign up for the mikoto ID Web App.

3




Sign up

User Page

Enter your Password

mikoto ID QR code


3. After registering for the web app, tap [Show QR code] on the user page. Enter your password, and once the QR code is displayed, tap [NEXT].

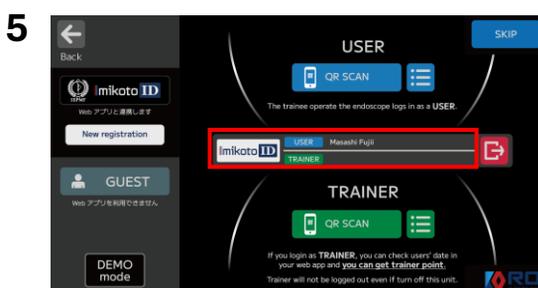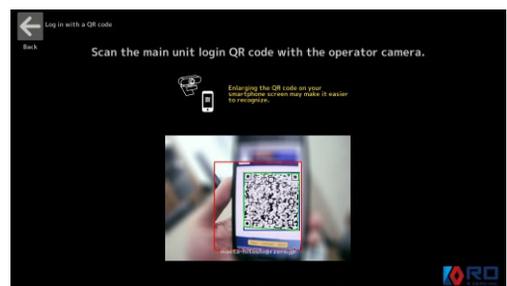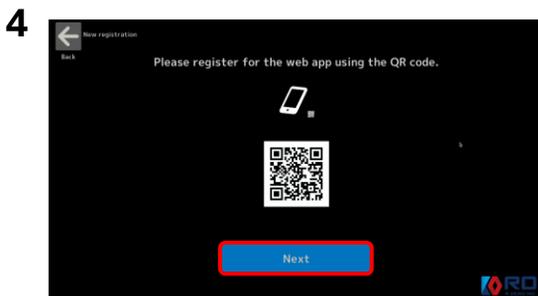

4. Scan the [mikoto ID QR code] displayed in the Web app using the camera connected to the mikoto device.


5. If your name appears in the red-framed User field, the user registration on the device is complete.

User login

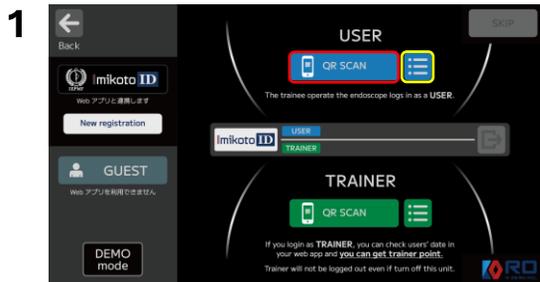

1 Tap [QR SCAN] under User to display the Web App sign-in QR code.

※If there is a previous user login history stored on the mikoto device, tap the list icon in the yellow frame and select the desired entry from the registration history.

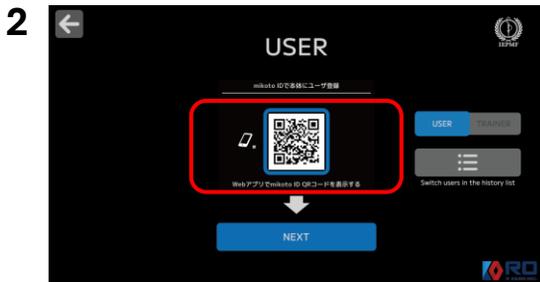


2. Scan the mikoto Web App registration QR code displayed on the mikoto device with your smartphone's camera to sign up for the mikoto ID Web App.


3 **Web app**

Sign up User Page Enter your Password mikoto ID QR code

3. After signing in to the web app, tap [Show QR code] on the user page. Enter your password, and once the QR code is displayed, tap [NEXT].

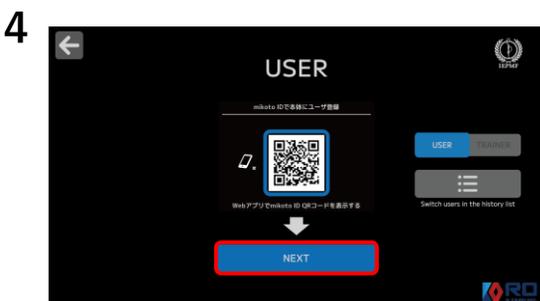

4. Scan the [mikoto ID QR code] displayed in the Web app using the camera connected to the mikoto device.

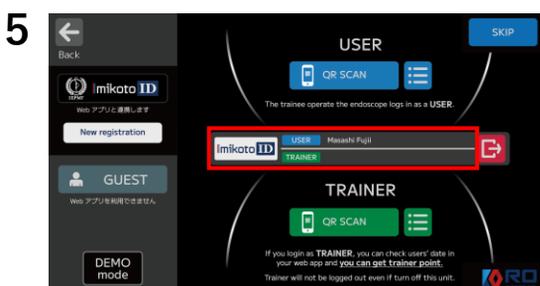

5. If your name appears in the red-framed User field, the user registration on the device is complete.

Trainer login



1. Tap [QR SCAN] in the Trainer menu to display the Web app sign-in QR code.

※If the Trainer is already registered on the mikoto device, tap the list icon in the yellow frame and select it from the registration history.



2. Scan the QR code displayed on the mikoto device using your smartphone's camera, then sign in to the mikoto ID Web app.

3

Sign up

User Page


Enter your Password

mikoto ID QR code

3. After signing in to the web app, tap [Show QR code] on the user page. Enter your password, and once the QR code is displayed, tap [NEXT].




4. Scan the [mikoto ID QR code] displayed in the Web app using the camera connected to the mikoto device.

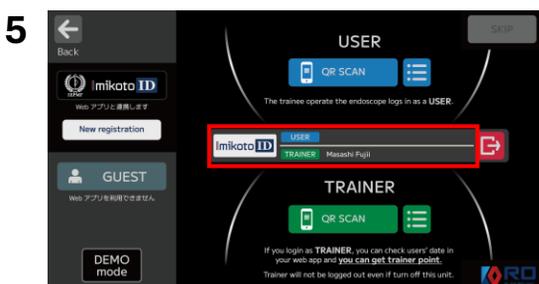

5. If your name is displayed in the red-framed Trainer area, the Trainer login to the device is complete.


Logout Procedure

1

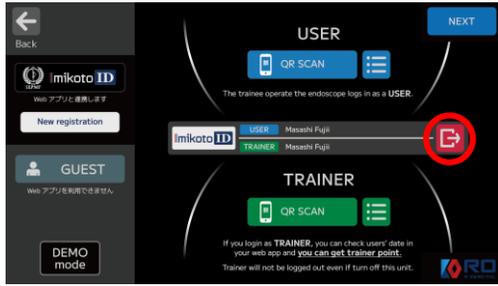

1. Tap the red-circled door icon.

2

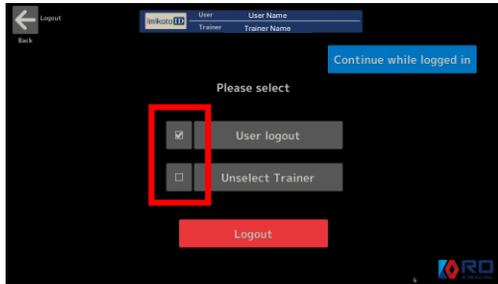

2. Select the items you want to log out from, then tap [Logout].

3



3. Logout is complete when the selected names no longer appear on the screen.

Data transfer

If the mikoto unit is not connected to the network, perform this step




1. Tap the [Data Transfer] icon

※The number of unsend data items is displayed in the upper-right corner of the icon.



2. The QR code containing the training data stored in the mikoto device will be displayed.

Scan it with your smartphone camera to transfer the training data to the Web app.

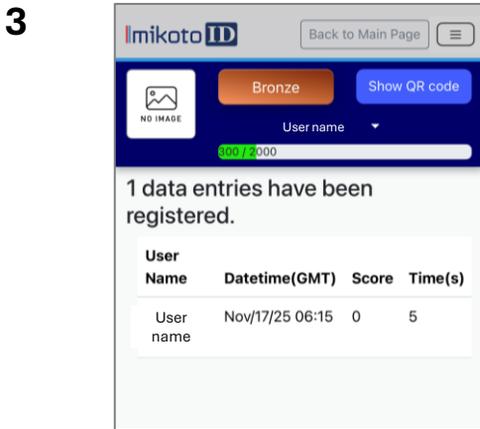

3. When 'Registration Complete' is displayed on the Web app screen, the training data transfer is complete.

New Registration

mikoto ID – web app

Sign up QR

①



mikoto ID – Web App / Sign Up

- ① Scan the sign-up QR code with your smartphone's camera to access the mikoto ID Web app and complete your user registration.
- ② Fields marked with a red indicator are required. Please fill in all mandatory items.
- ③ After entering all necessary information, check the box to agree to the Privacy Policy, then tap [Sign up] to complete your registration.

Sign in

mikoto ID – web app

Sign in QR



Scan the sign-in QR code with your smartphone's camera to access and sign in to the mikoto ID Web app.


User QR code

- ① Tap [Show QR code] on the mikoto ID Web app screen.
- ② Enter your password and tap [Show QR code] again.
- ③ The mikoto ID QR code will be displayed.

mikoto ID – Web app

status

Points are added for each training session, and your status rank is determined based on the total number of points earned.

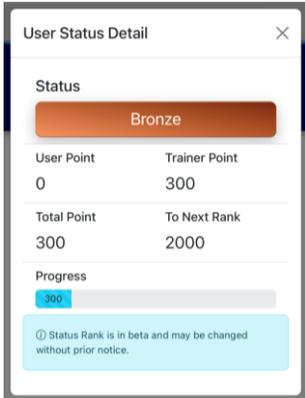



Show QR code
Display the user QR code.

user setting
mikoto ID QR code
Profile Edit
Sign out

menu

- Training History
- mikoto Devices
- Profile Edit
- Sign out

Point gauge
Your status will change depending on the number of points you have.


user / trainer view
Switch between User data and Trainer data.

Training Total Count
Total number of training sessions across all devices

Training Total Time
Total training time across all devices

mikoto model
By connecting with a mikoto unit, you will be able to view detailed information.

User / Training Data

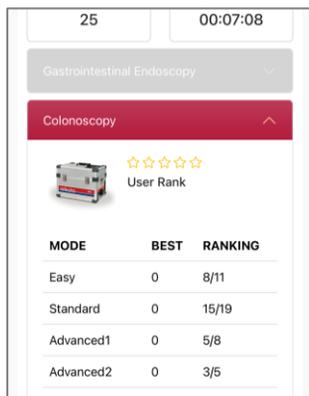


MODE	BEST	RANKING
Easy	0	8/11
Standard	0	15/19
Advanced1	0	5/8
Advanced2	0	3/5

By linking your mikoto ID with a mikoto unit, you can view your User Rank and Training Data. User Rank is displayed based on the achievement conditions for each device type.

Displayed Information
User Rank
Training Mode
Best Score
Ranking

Trainer / Data

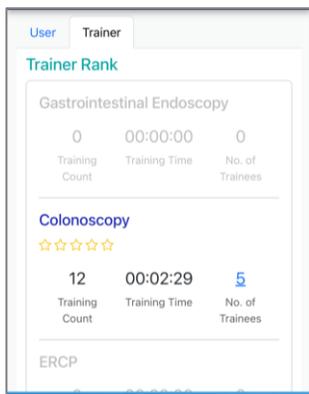

Your Trainees

Gastrointestinal Endoscopy

Colonoscopy Trainees: 5

Name	Best Score	Total Time
Yas Yam	-	00:01:00
Yasumasa Yamashita	-	00:00:10
5 0	100	00:00:29
55 66	100	00:00:02
Hitoshi Maeta	-	00:00:06

By registering a Trainer in the mikoto unit, the system will display the corresponding Trainer Data.

Displayed Information

- Trainee Training Total Count
- Trainee Training Total Time
- Trainee Registrant

(Tap to view detailed information about the trainee.)

※Please note that the User Interface design may change without notice.